candybabe

A MORE VIBRANT LIFE

USER MANUAL

Choose one:

Dutch English French German Italian Polish Slovak Spanish

www.candybabe.shop

SHINY KEGELS

Item: 1 Silicone Rope, 2 Silver color balls, 2 Golden color & magnetic balls

Material: Medical Grade Silicone+ABS

Specification: Silver color ball: 35MM,39g

Golden color & magnetic balls: 35MM,55g

Silicone rope: 150MM,39g

Instructions for use

For the first time, please use the silver color ball, they are lighter than the golden color and not magnetic thus decreasing the effort:

1. Before use, clean them following our “Maintenance” guidance
2. . Tight well the balls in the rope, then lie on your back and bend your knees, hold the tail end of the rope in your hand, and gently push the ball into the body, keeping the silicone rope exposed. After lying flat, relax and start exercise.
3. Get up gently, you can do squats or take the stairs to strengthen the exercise. The initial use can be 5-10 minutes, and then extend the exercise time after applicability, but do not exceed 30 minutes each time, otherwise the abdominal muscles will soar.
4. After training with a single ball (usually 10-15 days), you can use the double balls, and gradually increase to 30 minutes of training from 10 minutes a day.
5. After each use, clean them following our “Maintenance” guidance.

Principle of gravity: After the vaginal ball is put into the body, under the action of gravity, the ball will slide down, promote the automatic contraction of the muscles, and strengthen the training of the pelvis for a long time to achieve a tightening effect.

Maintenance

1. Correct maintenance product can effectively extend the life of the product.
2. Before and after each use of this product, please clean with warm water and mild soap or professional adult toys cleaner. Hard detergents, disinfectants, acetone or alcohol are NOT recommended to clean your Kegel balls.
3. If it has grooves or lines, be sure to clean these dead corners. You can use a cotton swab or cotton pad to carefully wipe them, but never use a toothbrush or brush to clean them.
4. Rinse all the foam on the toy under running tap water until there is some dryness in the friction. To dry your toy, please use a clean towel or a lint free paper towel (such as facial tissue and paper towel) by pressing lightly (note that you should never use ordinary paper towels to avoid breeding bacteria caused by residual paper scraps).
5. Store in a dust-free place and a cool dry area, to ensure that the product avoids direct sunlight, high temperature and humidity. Keep it out of children’s reach.

Warranty

Manufacturer warrants this device according to EU regulations for a period of 2 years, after the date of original purchase against defects due to faulty workmanship or materials. The warranty shall not therefore apply in the case of damages caused by improper use, wear or accidental events. Do not take it apart by yourself or by an unauthorized repair

shop, as your warranty will automatically expire.

Attention

1. This product is only suitable for adult use, it is non-contraceptive and not a medical product. The use of the product is strongly discouraged when pregnant. If you have a pacemaker, or if you have had surgery in the area where you want to use the product, consulting a doctor is recommended before using the product. Consult a doctor to get advice on how to use Kegel balls. This product is not suitable for anal sex, please use it for the intended use. Avoid sharing this device with others due to hygiene concerns.
2. Only water based lubricant is recommended to use with this device. Do not use massage oils or hand cream as lubricants for this device, as you will damage its silicone soft touch surface.
3. Please do not mix this product with others. Stop using this device if its tip is chipped or broken or has sharp edges. Stop use immediately if you feel unwell during use.
4. Please do not massage open wounds and throat areas.

Environment

1. To preserve the environment, we minimized the plastic package and used only recycled paper and recycled foam for packaging.
2. Please do not treat this toy as the household waste.
3. Please dispose of the toy and package in an environmental friendly way.

Disclaimer: Users of this device are doing so at their own risk. Neither Voluptas Group nor its retailers are responsible or assume any liability caused by the use of this product. This product doesn't claim to cure any disease or STI.

Candybabe is owned and managed by Voluptas Group, VAT: BG207374173 Address: 68 Rayko Daskalov street, Floor2, Office 8, Plovdiv 4000 Bulgaria. care@cadybabe.shop

This product meets EU safety, health and environmental protection requirements. Made in China

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